





FLORASEAL[®]

MICROBIAL SEALANT


PROFESSIONAL USE REFERENCE

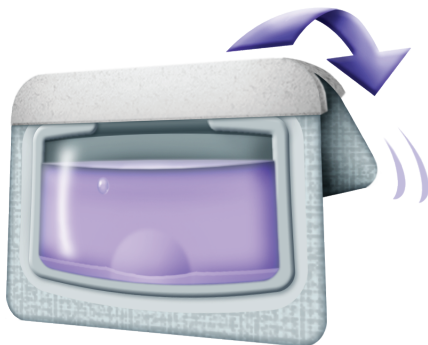

1. While holding the applicator with the applicator sponge facing upward, snap along the perforated line and fold over. Use shortly after opening the applicator.

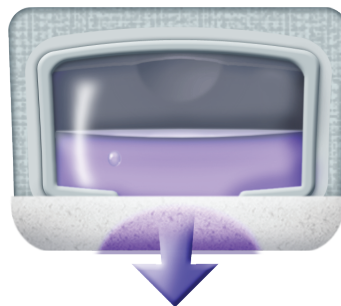

2. Turn applicator so sponge is facing downward. The flow is gravity-fed; **DO NOT SQUEEZE**. Paint an even, continuous layer over the entire area with no breaks between strokes. This may be accomplished with one pass of the applicator, attempting to avoid gaps. Allow the sealant to dry into a film.

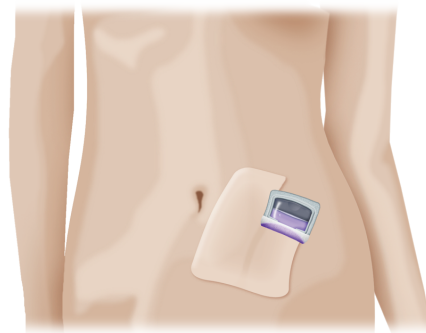

Medical Adhesive
Technologies

H.B. Fuller Medical Adhesive Technologies, LLC, 506 Pine Mountain Rd, Hudson, NC 28638, USA
Tel: +1.484.334.2929 · +1.866.579.4508 · www.hbfuller.com/medical
FloraSeal[®] is a registered trademark of H.B. Fuller Medical Adhesive Technologies, LLC.
©2025 H.B. Fuller Medical Adhesive Technologies, LLC. All rights reserved.

CE 0413

FSD-PUR1-2505