

# Applicators

## Wide range for personalised procedures



**The Drop-by-drop** devices are designed to allow accurate applications of Glubran® 2 to take advantage of its adhesive property to replace or reinforce sutures, to cover small areas, for haemostasis or for sealing. The product must be applied in a quantity of about one drop per cm<sup>2</sup> in order to achieve the best result.


**The Dispensing tip** for open surgery features a brush tip for applying a thin layer. Particularly suitable for dispensing Glubran® 2 to seal anastomoses and reinforce sutures.

**The Flexible and rigid laparoscopic catheters** are indicated for drip application of Glubran® 2 as a haemostatic, adhesive or sealant agent in laparoscopic surgical procedures.

**The nebulizers** with autonomous gaseous propulsion are designed to enhance all the properties of Glubran® 2, both in laparotomic and laparoscopic surgical procedures, by forming a thin film of product.

### Nebulizers

- Preparation is quick and easy
- These are independent single-use systems
- They do not require pressure regulators
- They do not connect to operating room air fittings or to CO<sub>2</sub> cylinders
- The inert propellant for nebulisation is non-toxic, biocompatible and has an autonomy of 8 minutes of continuous supply
- The outlet pressure is constant and controlled with no risk of air embolism

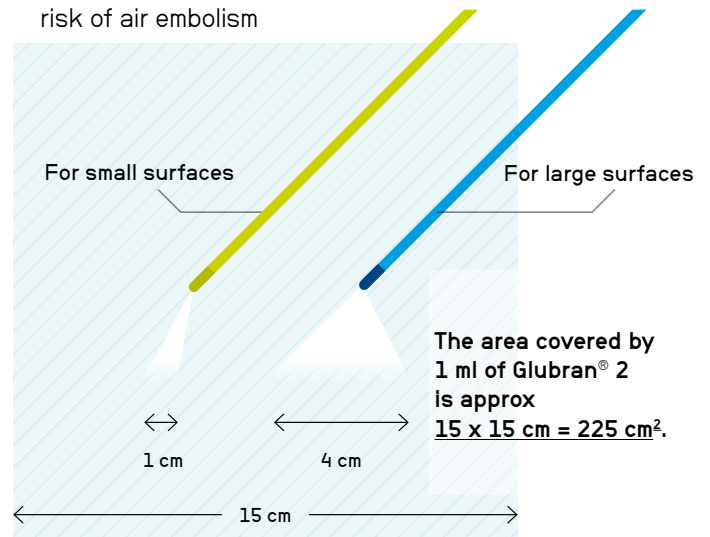

## Drop-by-drop control application device



G2-DCD-210-8T

## Thin film application devices



G-DT






## Laparoscopic catheters for drop-by-drop control application



G2-LPC FLEXIBLE



G2-LPC-RIGID RIGID

Nebulizers	APPLICATIONS	DISTANCE FROM TISSUE	NEBULIZING DIAMETER	PROPERTIES	RATIONALE
<b>NEBULIZER DEVICE</b> G2-NBT 		5-7 cm	4 cm	Flexible, for large surfaces, easy pointing, also suitable for open surgery	
<b>SMALL NEBULIZER DEVICE</b> G2-NBT-SMALL 	In laparoscopy thoracoscopy	2-3 cm	1 cm	Flexible, for small surfaces, easy pointing, also suitable for open surgery	SEALING
<b>RIGID NEBULIZER DEVICE</b> G2-NBT-RIG 		5-7 cm	4 cm	Rigid, for large surfaces	HAEMOSTASIS REINFORCEMENT
<b>SHORT NEBULIZER DEVICE</b> G2-NBT-SHORT 		5-7 cm	4 cm	Short, for large surfaces	AEROSTASIS BILIOSTASIS
<b>SHORT MINI NEBULIZER DEVICE</b> G2-NBT-SM-SHORT 	In open surgery	1 cm	1 cm	Short, for small surfaces	

## GLUTACK

### Laparoscopic atraumatic fixing device for hernia prostheses



GB-DS-30  
GB-DS-60

GB-DS SH 30  
GB-DS SH 60

Available in 2 sizes, 30 or 60 drops, depending on the type of procedure and size of the hernia mes

